

Table of Contents

Chapter 1. Overview

1.1 Introduction to the System	1/2
1.2 Indications	1/2
1.3 Symbols used in this Manual and the Device	2/2

Chapter 2. Safety Precautions

2.1 Overview	1/10
2.2 Treatment-Related Safety Procedures	2/10
2.3 Precautions when Moving the Device	3/10
2.4 General Precautions for Physicians, Staff and Patients	4/10
2.4.1 Precautionary items for HEALITE II prior to operation	4/10
2.4.2 Precautionary items during use	4/10
2.5 Precautions against Electrical Hazards	5/10
2.6 Precautions for Fire Hazards	6/10
2.7 International Standards Applied to HEALITE II	7/10
2.8 Safety Labels for HEALITE II	8/10
2.8.1 Warning label for electrical shock	8/10
2.8.2 Warning label for replacing the installed head	9/10
2.8.3 Warning label for moving the device	9/10
2.8.4 Prohibition label about no pushing	9/10
2.8.5 HEALITE II system nameplate label	10/10

Chapter 3. System Description

3.1 Overview	1/5
3.2 System Specification	1/5
3.3 Components of the HEALITE II	2/5
3.3.1 System main body	3/5
3.3.2 Arm & head	4/5
3.4 System Software	5/5
3.5 Essential performance	5/5

Chapter 4. Installation

4.1 Overview	1/11
4.2 Installation Component List	1/11
4.3 Conditions for Installation	2/11

4.3.1 Space requirements	2/11
4.3.2 Electrical requirements	3/11
4.3.3 Environmental requirements	4/11
4.4 Installation of the Device	5/11
4.4.1 STEP 1 : Components of the device	5/11
4.4.2 STEP 2 : Connecting the arm to the main system body	6/11
4.4.3 STEP 3 : Connecting the arm cable to the main system body	7/11
4.4.4 STEP 4 : Connecting the head to the arm	8/11
4.4.5 STEP 5 : Connecting the main power cable	10/11
4.4.6 STEP 6 : Final check-up and locking the system in place	10/11
4.5 Moving the Device	11/11
4.5.1 Moving the device in the treatment room	11/11
4.5.2 Moving the device to a remote location	11/11

Chapter 5. Operation

5.1 Overview	1/15
5.2 Checklist of Precautionary and Inspection Items	5/15
5.3 Operation of the HEALITE II system	6/15
5.3.1 STEP 1 : Supplying power	6/15
5.3.2 STEP 2 : Turning on the system	7/15
5.3.3 STEP 3 : Operating mode	8/15
5.4 Setup	10/15
5.5 How to use the Head	11/15
5.6 Turning off the System after Treatment	11/15
5.7 Intensity Setup (Engineering Purpose Only)	12/15

Chapter 6. Maintenance and Management

6.1 Overview	1/4
6.2 Cleaning the System Console	1/4
6.3 Troubleshooting	2/4
6.3.1 When the system fails to power up after pressing the main power button, please check the following items.	2/4
6.3.2 When the device is powered up but LED energy is not emitted.	2/4
6.4 Error Messages	3/4

6.5 Requests for Periodic after-sales service 4/4

Chapter 7. Warranty

8.1 Overview 1/3
8.2 Unpaid Services 1/3
8.3 Exclusion to unpaid service within the period of the warranty 2/3